MEDICON eG GERMANY · 47° 57' 31.4" N · 8° 46' 58.6" E

PRESTO

Bewahrung der haltenden Bänder und die Anbindung des SMAS
Preservation of the Retaining Ligaments and the SMAS tethering
La conservación de los ligamentos de retención y la conexión del SMAS
La conservation des ligaments de maintien et l'intégration de SMAS
La conservazione dei legamenti facciali e la connessione con SMAS

<https://ra.medicon.de>

Not all products illustrated in this documentation are available for sale in all countries. The products shown in this catalogue comply with the Medical Device Directive 93/42/EEC (MDD).

**Just scan the QR-Code
and you already
reach the latest
information!**

medicon®

Not all products illustrated in this documentation are available for sale in all countries.
Please contact your local distributor for more information.

Modelländerungen vorbehalten
Patterns are subject to change
Salvo modificaciones
Tous droits réservés des changements de modèle
Ci riserviamo la facoltà di cambiamenti nei modelli

Für Druckfehler übernehmen wir keinerlei Haftung
We are not liable for misprints
Declinamos toda responsabilidad por los errores de impresión que pueda contener este documento
Nous déclinons toute responsabilité en cas d'erreurs d'impression
Non ci assumiamo nessuna responsabilità per eventuali errori di stampa

Gedruckt in Deutschland
Printed in Germany
Impreso en Alemania
Imprimé en Allemagne
Stampato in Germania
Trykket i Tyskland

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The PRESTO technique is the technique that complies with patients' expectations which, according to scientific research, include naturalness, youthfulness, freshness and attractiveness. The PRESTO technique is furthermore a fast, secure and identity-preserving technique that ensures outstanding, long-lasting results.

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- Advantages**
- ▶ Atraumatic, blunt dissection of the tissue
 - ▶ Very fast mobilisation of large areas with little spreading forces
 - ▶ Injury of nerves, tendons and ligaments is virtually excluded
 - ▶ Identity-preserving and long-lasting result

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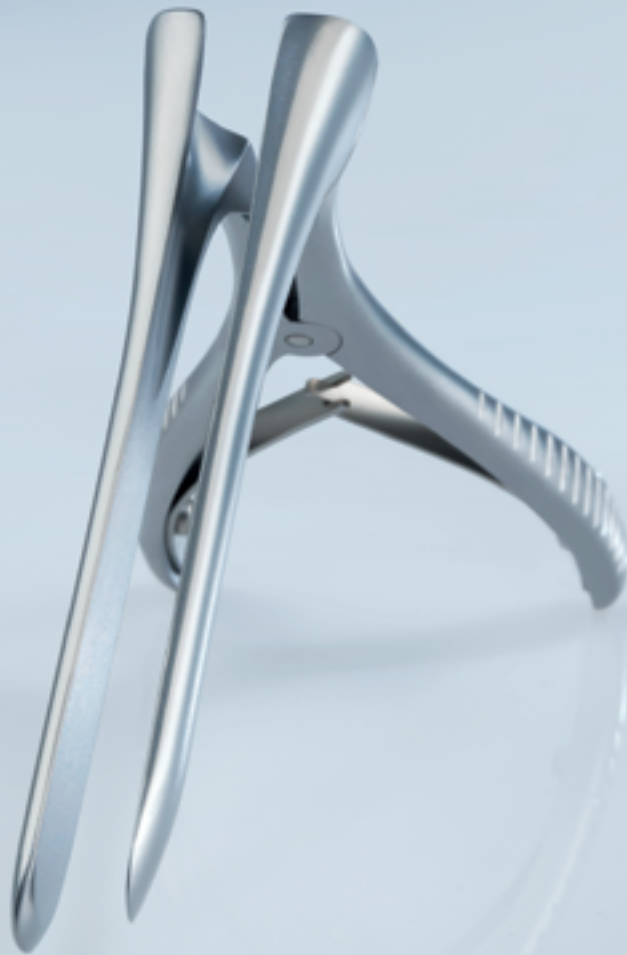
During the last 100 years, development of facelift techniques proceeded from the purely cutaneous technique through the SMAS technique (SMAS = Superficial Muscular Aponeurosis System) to the subperiosteal technique. By the beginning of the millennium, techniques in Mendelson's Layer 4 rounded off the plastic surgeons' portfolio. The decisive discovery in facelift technology was the discovery of the aponeurotic system in Layer 3, the so-called SMAS.

The SMAS serves as the support structure within the facial soft tissue. The fixation of the anterior facial oval, i.e. the mobile structure of the face, is held via the SMAS and can be fixated to the non-mobile, lateral facial fascia.

The initial efforts to dissect the SMAS to varying extents with opening of the retaining ligaments, also in the sense of a super-extended facelift, resulted in maximum force transfer at the dissected ends of the SMAS. Overextension at the basal structure and thus a dynamic sign of the operated face was almost inevitable and presented a great challenge for surgeons and patients alike.

The retaining elements, i.e. the combination of retaining ligaments and interlayer septae, must be preserved in order to achieve a dynamic force transmission via the SMAS. Thus, segmental force distribution is ensured, and overextension as well as a dynamic sign of the operated face is avoided.

Furthermore, the retaining elements guarantee the preservation of the personal identity which defines our facial appearance. The identity is protected by the preservation of the retaining element, thus counteracting alienation of the personality.

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© Copyright 06/2017, MEDICON eG, Tuttlingen, Germany