Segment||Distractor

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System for Shifting Alveolar Ridge Segments

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“HELENE MATRAS AWARD”

from the Austrian Cleft Palate Craniofacial Association

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The Medicon eG "Segment Distractor" system is used to move mandibular and maxillary alveolar ridge segments. The award-winning innovation combines modern surgical techniques with constructive design and enables individual modelling and anatomically correct shaping of the alveolar ridge. The system comprises the following components: serrated rail, drive module, distal and paranasal anchoring plates. It can be supplemented with titanium screws and the associated tool from our Osteosynthesis MINI System 2.0 for bone anchoring.

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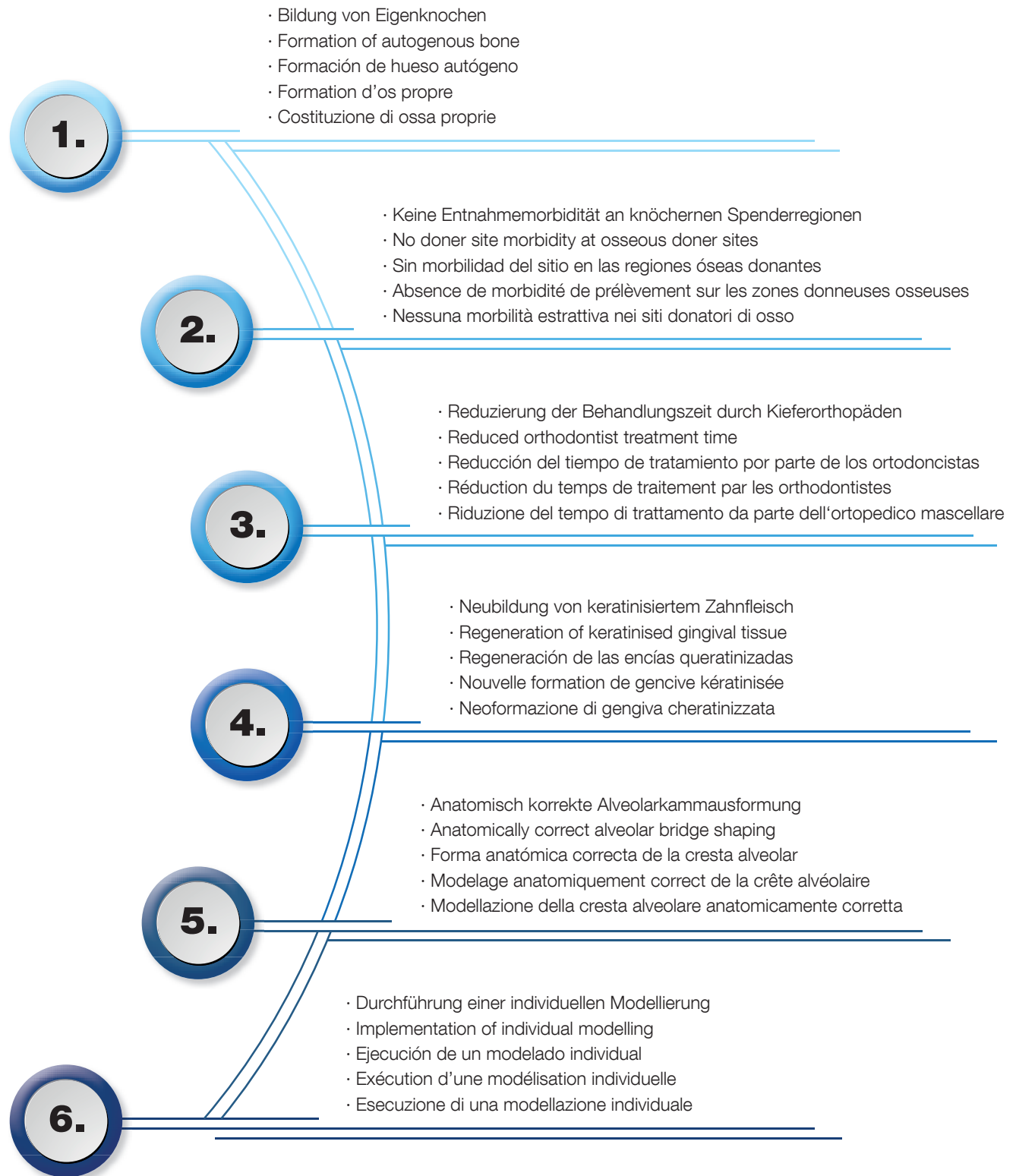
Prof. Dr. Dr. Wolfgang Zemann

Clinical Department of Oral and Maxillofacial Surgery. University Clinic for Dentistry and Oral Health, Graz

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The Medicon eG System "Segment Distractor" can be used for bilateral cleft lip and palate. A two-stage treatment is recommended for this. The latency between the two treatments is approx. 16 weeks.

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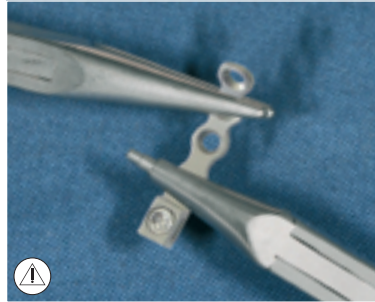
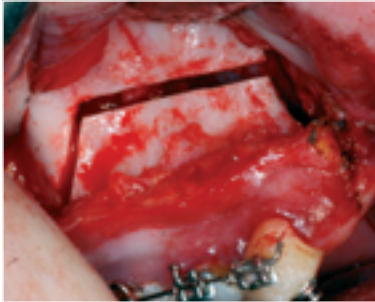
Surgical procedure:

- Preoperative planning using the plaster dental model.
- Implementation of segment osteotomy in usual manner.
- Sufficient mobilisation of the segment is recommended. A palatal pedicle must be maintained.

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The adjustment of the distal anchoring plate ① to the zygomatic process of the maxilla on the side of the planned segment distraction is carried out with two plate bending forceps (see page 13).

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For the fixation of the distal anchoring plate ① in the area of the zygomatic process of the maxilla, we recommend titanium Philips screws with the associated tool from the Medicon eG Osteosynthesis MINI System 2.0 (see pages 12-13).

- In all cases, sufficient stability must be ensured.

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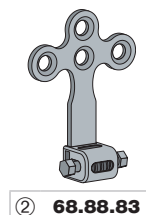
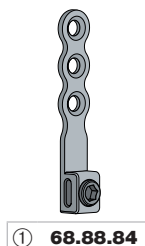
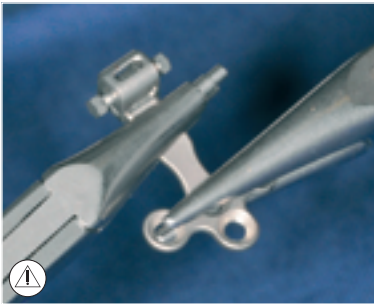
The adjustment of the drive module ② to the mobile osseous segment is carried out with two plate bending forceps (see page 13).

Caution: Dental roots!

➤ The plate should later be fixed so that the drive module does not compromise the gingival tissue (gum).

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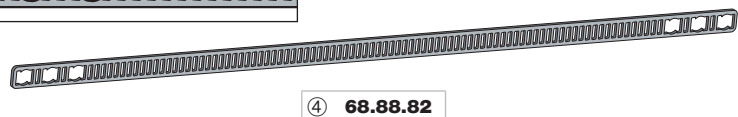
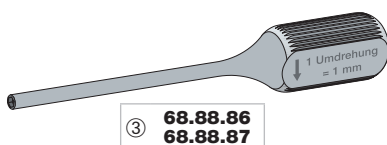
Insert the serrated rail ④ with the inserted drive module ② into the distal anchoring plate ① and slide to one of the three slots (see figure below). There are three options corresponding to the anatomical conditions. Next, tighten the hexagonal fixing screw using a screwdriver (see page 13) so that the plate and serrated rail are firmly anchored.

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For the fixation of the drive module ② to the mobile osseous segment, we recommend titanium Philips screws with the associated tool from the Medicon eG Osteosynthesis MINI System 2.0 (see pages 12-13).

➤ In all cases, sufficient stability must be ensured and the drive module must not compromise the gingival tissue (gum).

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The adjustment of the paranasal anchoring plate ⑤ to the zygomatic process of the maxilla on the opposite side of the planned segment distraction is carried out with two plate bending forceps (see page 13).

➤ The plate must be bent carefully, avoiding excessive deformation. The plates must not be bent back and forth more than two to three times.

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Place the paranasal anchoring plate ⑤ on the serrated rail and move it to the defined position. For the fixation of the paranasal anchoring plate ⑤, we recommend titanium Philips screws with the associated tool from the Medicon eG Osteosynthesis MINI System 2.0 (see pages 12-13).

➤ In all cases, sufficient stability must be ensured.

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The adjustment of the second distal anchoring plate ① to the zygomatic process of the maxilla on the opposite side to the planned segment distraction is carried out with two plate bending forceps (see page 13).

➤ The plate must be bent carefully, avoiding excessive deformation. The plates must not be bent back and forth more than two to three times.

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Place the second distal anchoring plate ① onto the opposite side of the serrated rail ④ and slide it to one of the three predetermined slots (see figure below). There are three options corresponding to the anatomical conditions. Next, tighten the hexagon fixing screw using a screwdriver (see page 13) so that the plate and serrated rail are firmly anchored.

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For the fixation of the second distal anchoring plate ① to the zygomatic process of the maxilla, we recommend titanium Philips screws with the associated tool from the Medicon eG Osteosynthesis MINI System 2.0 (see pages 12-13).

➤ In all cases, sufficient stability must be ensured.

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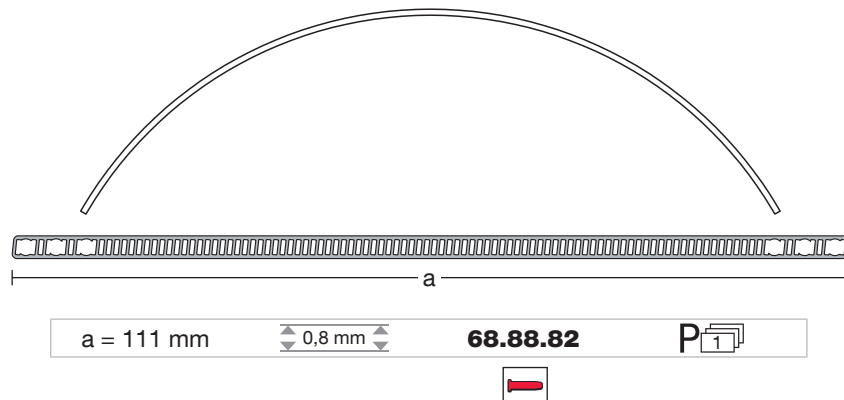
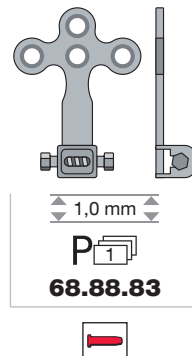
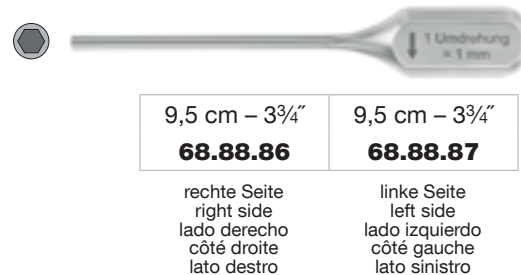
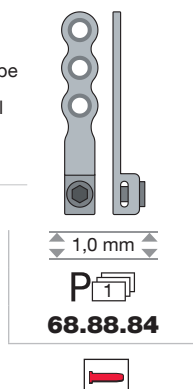
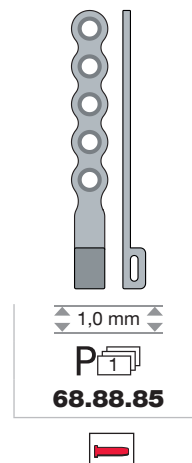
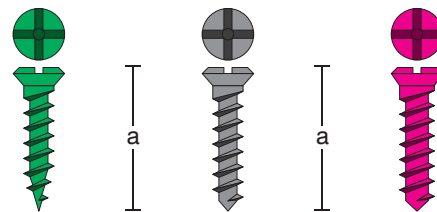
The "TraceID"-pins as well as the four identification specifications on the screw cartridges guarantee a reliable traceability of implants.

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	Self-drilling P ₁₀	Self-tapping P ₁₀	Emergency screw P ₁₀
a	Ø 2,0 mm	Ø 2,0 mm	Ø 2,3 mm
5,0 mm	68.94.15A		
5,5 mm		68.94.01A	68.94.09A
7,0 mm	68.94.17A	68.94.02A	
9,0 mm	68.94.19A	68.94.03A	

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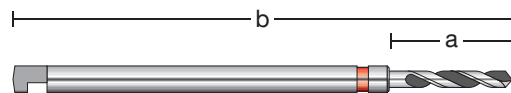
— MEDICON —
implantArt 2.0 MINI
 — OSTEOSYNTHESIS —

Maxillofacial System
Fracture System
Fracture Locking System

Spiralbohrer
 Twist drill
 Broca espirale
 Foret hélicoïdal
 Perforatore elicoidale

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
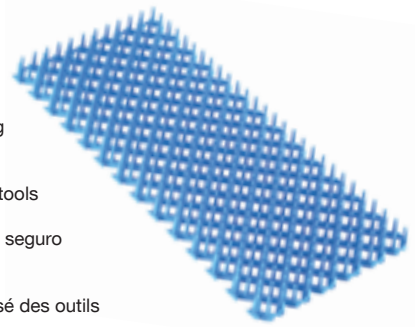
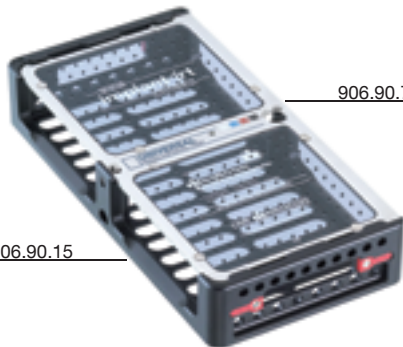
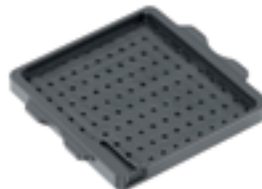
Segment||Distractor

Clinical Advice:

- After implementation of the segment osteotomy any bony interferences on the segment and/or alveolar ridge must be smoothed before the segment distractor is installed.*
- When positioning the drive module avoid damage of dental roots during fixation of the plate. In case of collision the drive module must be repositioned and refixed.*
- At least 2 screws or more are needed for a stabilised fixation.*
- An injured dental root is regenerating itself by creating cellular cement. (see image below)*
- After wound-closure the distraction device is located orally between attached and mobile gingiva.*
- When removing the segment distractor, an osteoplasty may necessary to close the gap between the segments and/or the nasal floor. Bone can be harvested intraorally from the zygomatic process of the maxilla.*
- In the case of bilateral cleft palate, a two-stage treatment is recommended.**

Source of informations: * Prof. Dr. odont. Birte Melsen, Århus | ** Prof. Dr. Dr. Wolfgang Zemmann, Graz

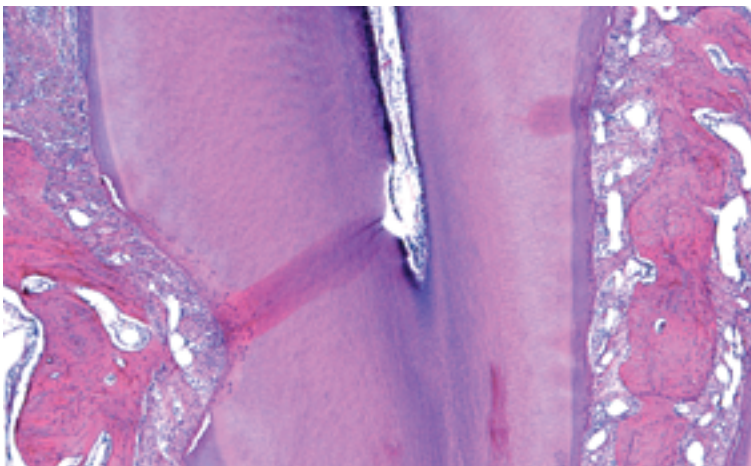Medicon eG

Gänsäcker 15 · D-78532 Tuttlingen

Tel.: +49 (0) 74 62 / 20 09-0

Fax: +49 (0) 74 62 / 20 09-50

E-Mail: sales@medicon.de

Internet: www.medicon.de

Germany

The products shown in this catalogue comply with the Medical Device Directive 93/42/EEC (MDD).

Not all products illustrated in this documentation are available for sale in all countries. Please contact your local distributor for more information.

Gedruckt in Deutschland
Printed in Germany
Impreso en Alemania
Imprimé en Allemagne
Stampato in R.F.G.
Trykkt i Tyskland

Modelländerungen vorbehalten
Patterns are subject to change
Salvo modificaciones
Tous droits réservés des changements de modèle
Ci riserviamo la facoltà di cambiamenti nei modelli

453.10.00

© Copyright 05/2023, MEDICON eG, Tuttlingen, Germany